**Read the instructions carefully before use**

### Valid for 1 patient

- 1 syringe of dental whitening gel, NATURAL<sup>+</sup>™, 6% stabilized hydrogen peroxide, 7ml
- 2 application trays custom made for you by your dental surgeon
- 1 application tray case
- 1 toothbrush in a case
- 1 syringe of pH neutral stain-removing gel for use exclusively by professionals, Powerprep<sup>+</sup>™, 1.2g
- 1 tip
- 1 gentle stain-removing toothpaste, Powerprep<sup>+</sup>™, 30ml
- 1 tube of nourishing and soothing treatment, #5, 5ml
- 1 container with dental floss and mirror
- 1 shade guide

### Precautions:

Store in a cool place, away from sunlight.

Keep out of reach of children.

Do not swallow. If swallowed, consult a doctor.

Pregnant or breastfeeding women must consult their doctor before use.

Do not use under the age of 18.

Can only be sold to a dentist.

Teeth whitening procedure is an act dedicated to the dental practitioners only.

Specialist only can make a diagnosis of your teeth before use; required before every teeth whitening procedure.

Professional only is allowed to give you instructions for use after a first treatment performed at the clinic and inform you about eventual contraindications.

Products to use under the strict supervision of the dentist.

Avoid contact with eyes. In case of accidental contact, rinse thoroughly and consult a doctor.

### Warnings:

Consumers of alcohol and/or tobacco should refrain from smoking and/or drinking as much as possible throughout the duration of the NATURAL<sup>+</sup>™ dental whitening gel treatment, in order to receive optimal results. Do not apply the NATURAL<sup>+</sup>™ dental whitening gel to teeth with metal amalgams.

*Reminder: The products included in this kit have to be used under the strict supervision of a dentist only.*

### Instructions:

#### For dentists (in the clinic):

1. Check the starting shade with a shade guide and mark it.
2. With this patient box comes a box of 6 syringes each containing 7ml of NATURAL<sup>+</sup>™ dental whitening gel, 6% stabilized hydrogen peroxide. One of these syringes should be added to this kit.
3. Make unique thermoforming application trays for your treatment at the clinic and the patient's use at home and place them in the application tray case. It is not recommended to make a reservoir in the trays
4. Use the trays and apply a small drop of the neutral pH stain-removing gel, Powerprep<sup>+</sup>™ into each tooth location after positioning the tip on the syringe.  
Apply the gel on both buccal and lingual sides.  
Leave the trays in the patient's mouth for 15 minutes; then, remove them and rinse patient's mouth.  
Rinse the trays, dry them thoroughly and reuse them for the teeth whitening procedure.
5. Rotate the syringe plunger anticlockwise to unlock the dispenser.

Fig. 1

Fig. 2

Apply the NATURAL<sup>+</sup>™ dental whitening gel by pressing on the syringe plunger.


A dispenser built into the mechanism releases the required quantity for each tooth.

Rotate the syringe plunger clockwise to lock it and avoid unintentional use. (see Fig. 1,2)

6. Place the application trays on the teeth.  
It is not necessary to close the jaws more firmly in order to obtain better results.
7. Leave them in the mouth for an hour to an hour and a half.
8. Remove the application trays, and then rinse the mouth.  
Thoroughly rinse and dry the application trays and return them to their case, for the next session performed at home by the patient, according to the dentist's instructions.
9. Apply the nourishing and soothing gel to the teeth and gums, massaging steadily to avoid residual sensitivity.
10. Check the achieved shade with a shade guide and compare with the starting shade.

#### **For patients (at home):**

1. Use the shade guide and mark with a cross the starting shade.
2. Take the toothbrush provided with the kit and brush the teeth thoroughly for 3 minutes using the gentle stain-removing toothpaste, **Powerprep+™**, provided in the kit.  
Complete this stage by using the dental floss and pay particular attention to the spaces between the teeth.
3. Take the application trays given to you by your dental surgeon.
4. Rotate the syringe plunger anticlockwise to unlock the dispenser.

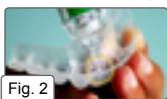A dispenser built into the mechanism releases the required quantity for each tooth.  
Rotate the syringe plunger clockwise to lock it and avoid unintentional use. (see Fig. 1,2)

5. Place the application trays on the teeth.  
It is not necessary to close the jaws more firmly in order to obtain better results.
6. Leave them in the mouth for an hour to an hour and a half.
7. Remove the application trays, and then rinse the mouth.  
Thoroughly rinse and dry the application trays and return them to their case, for the next session performed at home by the patient, according to the dentist's instructions.
8. Apply the nourishing and soothing gel to the teeth and gums, massaging steadily to avoid residual sensitivity.
9. Repeat this full process daily, as instructed by your dentist.
10. Check the achieved shade with the shade guide and compare with the starting shade.

#### **Ingredients:**

##### ■ **Dental whitening gel NATURAL<sup>®</sup>™ / Activator, 7ml**

Gel: Aqua, Hydrogen Peroxide, Silica, PE6-8

Activator: Aqua, Glycerin, Carboxymethylcellulose, Potassium Nitrate, Sodium Hydroxide, Sodium Fluoride, Dead Sea Salt (natural), CI 14720

##### ■ **Neutral pH stain-removing gel, Powerprep+™, for use exclusively by professionals, 1.2g**

Distilled Water, Citric Acid, Potassium Nitrate, Fumed Silica, Pigments

##### ■ **Gentle stain-removing toothpaste, Powerprep+™, 30ml**

Sorbitol 70%, Water (Aqua), Hydrated Silica, Glycerin, Cocamidopropyl Betaine, Sodium Citrate, Sodium Monofluorophosphate, Cellulose Gum, Sodium Bicarbonate, Citric Acid, Titanium Dioxide, Flavour (Supplement), Sodium Saccharin, Sodium Methylparaben, Sodium Propylparaben.

##### ■ **Nourishing and soothing treatment, #5, 5ml**

Aqua, Potassium Nitrate, Sea salt, Polysorbate 20, Cellulose Gum, Aroma, Sodium Methylparaben, Potassium Sorbate, Dicalcium Phosphate, Eugenol, Eugenia Caryophyllata, Sodium Fluoride, Isoeugenol

■ **Store in a cool place, away from sunlight**

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