

 Read the instructions carefully before use

Home Kit Refill

Each patient receives an individual box containing:

- 1 gentle stain-removing toothpaste, Powerprep+TM, 30ml
- 1 syringe of dental whitening gel, NATURAL+TM, 6% stabilized hydrogen peroxide, 10ml

Precautions:

Store in a cool place, away from sunlight.

Keep out of reach of children.

Do not swallow. If swallowed, consult a doctor.

Pregnant or breastfeeding women must consult their doctor before use.

Do not use under the age of 18.

Can only be sold to a dentist.

Teeth whitening procedure is an act dedicated to the dental practitioners only.

Specialist only can make a diagnosis of your teeth before use; required before every teeth whitening procedure.

Professional only is allowed to give you instructions for use after a first treatment performed at the clinic and inform you about eventual contraindications.

Products to be used under the strict supervision of a dentist.

Avoid contact with eyes. In case of accidental contact, rinse thoroughly and consult a doctor.

Warnings:

Consumers of alcohol and/or tobacco should refrain from smoking and/or drinking as much as possible throughout the duration of the NATURAL+TM dental whitening gel treatment, in order to receive optimal results.

Do not apply the NATURAL+TM dental whitening gel to teeth with metal amalgams.

Reminder: The products included in this kit have to be used under the strict supervision of a dentist only.

Instructions:

1. Prior to the procedure, brush your teeth thoroughly for 3 minutes using the gentle stain-removing toothpaste, Powerprep+TM.
2. Take the application trays given to you by your dental surgeon.
3. Rotate the syringe plunger anticlockwise to unlock the dispenser.



Apply the NATURAL+TM dental whitening gel by pressing on the syringe plunger, a dispenser built into the mechanism releases the required quantity for each tooth.

Rotate the syringe plunger clockwise to lock it and avoid unintentional use.

4. Place the application trays on the teeth. It is not necessary to close your jaw more firmly in order to receive a better result.
5. Leave them in the mouth for an hour to an hour and a half.
6. Remove the application trays, and then rinse your mouth.
7. Thoroughly rinse and dry the application trays and keep them in a dry, clean place until the next session.
8. Repeat this process as instructed by your dentist.



Store in a cool place, away from sunlight

LOT



Meodental Ltd., 14 Wallenberg St., Ramat Hachayal I.Z, Tel Aviv, Israel, 6971911

www.meodental.com | info@meodental.com