



**NATURAL<sup>+</sup>**<sup>TM</sup>

**Home Bleaching**



**NATURAL<sup>+</sup>**<sup>TM</sup>



**Home Bleaching**

Only **1** hour/day

Consults Your Dentist



## Home Bleaching

**NATURAL+™** Home Kit is a self administering kit that patients can obtain from the dental clinic after consultation.

The **NATURAL+™** Home Kit ensures optimum results from the comfort of the patient's home when combined with the **NATURAL+™** Professional application.

The **NATURAL+™** Home Kit also contains our exclusive **NATURAL+™** Gel which is safe and reliable and far more efficient and effective than any other product without the side effects or sensitivities.

### Application

1. Take the mouth trays (supplied by dentist)
2. Uniformly apply the **NATURAL+™** dental bleaching gel by repeatedly pressing the syringe piston once for each tooth cast; a dispenser included into the mechanism releases the required quantity with each press
3. Put the mouth trays into the mouth and apply gentle pressure
4. Leave in for one hour (may vary according to the clinical application)
5. Remove trays and rinse mouth
6. Rinse trays and store
7. Enjoy a whiter smile!

**Repeat daily according to Dentist's instructions**



**More Information :** XP PROFESSIONAL COMPANY LIMITED

66/9 MOO.2 SOI.LADPRAO 71, LADPRAO RD., LADPRAO, LADPRAO

BANGKOK 10230 TEL.0-2917-3396 FAX.0-2917-2297 EMAIL:xp\_prof@yahoo.com