

  
**NATURAL<sup>+</sup>**

**In-Clinic Bleaching**



**30 Min**

**One Cycle Only**

**Activated Bleaching Gel By**

**LED LAMP**

The First One and The Only One Bleaching Products By Using

**LED bleaching Lamp Technology**

Which is recognized as **SAFETY** technology for Medical Treatment

**Consults Your Dentist**



## In-Clinic Bleaching

The **NATURAL+™** Professional Kit is intended for in clinic application by a Dentist.

The **NATURAL+™** Professional Kit contains our exclusive **NATURAL+™** formula, our highly efficient and safe to use whitening solution plus all the requirements to perform the required procedure and produce effective and lasting results

The **NATURAL+™** Professional Kit is different from others that ONLY **30 minutes** treatment with **ONE cycle** only. No need the repeated gel application for 3-4 times like other systems. And that resulted as PATIENTS comfortable, no need to stay on the dental chair for 2 hours. While **NATURAL+™** gives the 100% EFFECTIVE results.

Before Bleaching, it is very important to let YOUR DENTIST check your oral health and also the original teeth color.

### **DIRECTION - Easy 5 Steps as following: (Pre-cleaning before bleaching)**

1. After clean the teeth. Apply the neutral pH stain-removing gel (POWER PREP+) and leave its for 10 minutes..
2. Rinse, then aspirate the surplus water. Apply the GINGIVAL PROTECTION (Ensure beforehand that the gums are completely dry).
3. Apply the bleaching gel for 30 minutes with LED LAMP (Ensure beforehand that the gels already mixed at least 10 minutes in advance.).
4. Remove the gel, Rinse then aspirate the surplus water . Remove the GUM PROTECTION. Then Using Rubber CUP for teeth cleaning with Prophylactic Paste enriched with Amorphous Calcium Phosphate
5. Last one, Apply the nourishing and soothing treatment to the teeth and gums, massaging steadily. Repeat every one hour until finished the gel.

## Enjoy Your Brighter Smile!!!

**More Information : XP PROFESSIONAL COMPANY LIMITED**

66/9 MOO.2 SOI.LADPRAO 71, LADPRAO RD., LADPRAO, LADPRAO BANGKOK 10230

TEL. 0-2917-3396 FAX. 0-2917-2297 EMAIL: xp\_prof@yahoo.com